Instructions For Use:

RHEIN RECOVERY BRA *RECOVER IN COMFORT*

- Rhein Recovery Bras have been designed with internal pockets to hold Rhein GEL-O's to aid in applying hands-free cold therapy for post surgical recovery.
- During the post surgery recovery phase, alternating between the two bras is recommended for GEL-O's cold compress therapy. After each cold therapy session, gently step out of damp bra and into dry one.
- It is recommended to step in and out of the Rhein Recovery Bras to avoid reaching overhead.
- Loosen bra straps to fit GEL-O's into the pockets, then tighten straps for compression after each GEL-O's cold therapy session. Continue to adjust the bra straps as swelling decreases.
- Rhein Recovery Bras are machine washable. See inside garment for washing instructions.

RHEIN GEL-O'S *COLD THERAPY*

- Rhein GEL-O's have been designed to provide cold therapy for post surgical recovery to reduce swelling, bruising and pain.
- Freeze GEL-O's for two hours prior to applying cold therapy. GEL-O's should only be applied to the relief area for the doctor recommended time of 20 minutes.
- GEL-O's cold therapy should help, not hurt. If the cold therapy feels uncomfortable, do not apply as often.
- CAUTION: Do not place GEL-O's directly against skin. Do not apply directly to open wounds, sensitive or insensitive skin. Use only as intended. Discard if punctured. Do not swallow contents. Non-toxic.
- CARE: GEL-O's can be cleaned with mild soap, detergent or alcohol. Do not place any heavy objects on top of GEL-O's. Bend GEL-O's at tabs to avoid possible damage.

FOR MORE INFORMATION AND STEP-BY-STEP
INSTRUCTIONAL VIDEOS, PLEASE VISIT

www.RheinCosmeticSurgerySolutions.com

Developed In Coordination With Robert D. Rehnke, M.D. & ReCon Surgical.

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